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LAVANID® - Wundgel V+

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wounds and to moisten bandages and wound dressings with preservative.

TISSUE TOLERANCE

Experimental and clinical studies have shown that the combination of Ringer's solution with 0.04% polyhexanide is very well tolerated by tissue. Clinical trials have shown that it does not negatively affect the formation of healthy new tissue. There are no local signs of irritant conditions. Use of this product is pain-free for patients.

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INFORMATION ABOUT SHELF-LIFE

The expiry date can be found on the container. Do not use after the expiry date!

INSTRUCTIONS FOR USE

LAVANID® - Wound Gel V+

DESCRIPTION AND PRODUCT PROFILE

The surface of chronic wounds is generally made up of dead and necrotic tissue, and is also colonised with bacteria. For wound treatment to be successful the wound surface needs to be removed, e.g. through surgical débridement. Wound irrigation and moistening are a necessary part of this process. Chronic wounds exhibit dysfunctional wound healing processes. This is why treatment with preparations that are very well tolerated by the tissues is indicated.

LAVANID® Wound Gel V+ is a sterile, preserved Ringer's solution-based wound gel which stands out due to its very good tissue tolerance. Polyhexanide 0.04% is used as a preservative. Polyhexanide is evaluated as the agent of choice for chronic wounds and burns. In addition to cleaning and moistening wounds, LAVANID® Wound Gel V+ is also appropriate for preservative dressings, e.g. for wounds colonised with multiresistant bacteria. Preservative dressings reduce the risk of carrying the bacteria from the wound to the patient's environment and they protect the wound from bacteria getting in from the outside.

COMPOSITION

Ringer's solution (isotonic electrolyte solution made up of sodium chloride, potassium chloride, calcium chloride 2 H₂O, water for injection), glycerin, macrogol, hydroxyethylcellulose, polyhexanide 0.04%.

INDICATIONS

LAVANID® Wound Gel V+ is used to moisten and clean

DISPOSAL INFORMATION

Dispose of unused products or waste material in accordance with the specific national regulations.

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