Delta-Solo® Cast Shoe

Delta-Solo® Sole

Delta-Solo® Straps

- EN Universal cast shoe
- DE Universal Gehschuh
- FR Semelle de marche universelle
- ES Zapato para pies escayolados – Talla única
- PT Sapato para pés imobilizados com moldes – Tamanho único
- IT Scarpa da gesso universale
- NL Universele gipsschoen
- SV Universal gipssko
- NO Gipssko, universal
- DA Universel gips-sko. Såle og stropper
- FI Kipsikenkä, sopii molempiin jalkoihin
- ZH 萬用石膏鞋
- AR ماع ساقم يبط اءاح

EN DELTA-SOLO® CAST SHOE / DELTA-SOLO® SOLE / DELTA-SOLO® STRAPS

Delta-Solo® is a pre-formed plastic sole with heel cup used for attachment to walking casts.

Intended use and indications

Delta-Solo® is intended to be used to protect casts and as a walking aid.

Directions for Use:

Delta-Solo® is intended for professional use. It is for single patient use. Delta-Solo® Cast Shoe, Delta-Solo® Sole and Delta-Solo® Straps can be used in various ways. Delta-Solo® Cast Shoe (A) is removable, Delta-Solo® Sole (B) can be permanently incorporated into the cast. Delta-Solo® Straps are available separately to add different lengths of straps if required.

For all applications, the initial steps 1–3 should be followed in preparation for fixation.

1. Apply cast as required with any casting material.
2. Flat Cast Sole – As with all rockers and boots, it is important to apply Delta-Solo® against a flat sole. A slab may therefore be required to achieve this.
3. Check size – Place the heel stop at the heel and check for length. If necessary, the toe portion can be trimmed with scissors.

A toe platform is frequently required for Rheumatoid Arthritis patients, in which case, the toe portion should be trimmed to the length of the big toe.

(A) Removable with Delta-Solo® Cast Shoe:

1. Secure and tighten the three pre-attached touch and close straps around the ankle and the foot.
2. The pre-attached touch and close straps Delta-Solo® Straps can be removed, changed around or replaced with other sizes of Delta-Solo® Straps, available separately.

Tighten straps to ensure a stable fit.

(B) Permanent Fixation with Delta-Solo® Sole:

Delta-Solo® Sole can be incorporated permanently into the cast by applying the cast through and over the following areas:

1. Heel Stop
2. Instep Channel

Cleaning and care

The Delta-Solo® Cast Shoe and Delta-Solo® Sole may be cleaned using a damp cloth and detergent-based solution. Remove the shoe from the cast before cleaning to avoid wetting the cast.

Notice:

Any serious incident that has occurred in relation to this device should be reported to the BSN medical Inc. (BSN medical GmbH within the European Union) and the competent authority of your state.

DE DELTA-SOLO® CAST SHOE / DELTA-SOLO® SOLE / DELTA-SOLO® STRAPS

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