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Cutimed® Sorbact® Hydroactive

Produktbeschreibung

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Cutimed® Sorbact® Hydroactive

Product description

Cutimed® Sorbact® Hydroactive is a non-adhesive absorbent gel dressing with bacteria binding action. Cutimed® Sorbact® Hydroactive consists of a highly absorbent hydropolymer matrix with a Sorbact mesh (Sorbact acetate fabric coated with dialkylcarbamoylchloride – DACC) and is covered by a semipermeable polyurethane film.

Cutimed® Sorbact® Hydroactive is available in 5 sizes:
7 x 8.5 cm, 14 x 14 cm, 19 x 19 cm, 24 x 24 cm and 14 x 24 cm.

Indications

Cutimed® Sorbact® Hydroactive is indicated for contaminated, colonised or infected exuding wounds, such as chronic wounds (venous and arterial ulcers, diabetic ulcers, and pressure ulcers), postoperative dehiscent wounds and traumatic wounds.

Contraindications

Do not use Cutimed® Sorbact® Hydroactive on dry wounds or cavity wounds. Do not use on patients with known sensitivity to the dressing components.

Warnings and Precautions

Cutimed® Sorbact® Hydroactive is packaged for single use. Do not re-use or re-sterilise as there is a risk of transmission of body fluids or contaminated tissue between patients. Do not use if the pouch is opened or damaged. Discard any open dressings. Frequency of dressing change of Cutimed® Sorbact® Hydroactive is dependent upon the wound condition and exudate level. Cutimed® Sorbact® Hydroactive can remain in place for up to 4 days.

When in contact with wound exudates, the highly absorbent hydropolymer matrix changes from yellow to brown. This indicates the dressing has become saturated and should be changed when exudate absorption reaches a maximum of 1cm from the edge of the hydropolymer.

Cutimed® Sorbact® Hydroactive is suitable for use in combination with compression therapy for contaminated, colonised or infected exuding wounds at low pressure.

Cutimed® Sorbact® Hydroactive can not be cut to shape. Avoid overlapping two or more Cutimed® Sorbact® Hydroactive dressings.

Do not use Cutimed® Sorbact® Hydroactive in combination with ointments or creams as they may interfere with the dressing's efficacy.

Directions for use

- Clean the wound as usual.
- Select an appropriate dressing according to wound size. For best results, make sure Cutimed® Sorbact® Hydroactive overlaps the wound margins by at least 2 cm.
- Remove the protective film and apply the green plain (unprinted) side of the dressing to the wound.
- If required, use a bandage for fixation. Cutimed® Sorbact® Hydroactive is compatible with gauze bandages, elastic gauze bandages, transparent film dressing and tapes for border fixation.
- Cutimed® Sorbact® Hydroactive should be changed as often as dictated by the wound condition. As with all absorbent dressings, monitoring is required to ensure the dressing does not dry out and adhere to the wound. The dressing should be changed when it becomes crusty or soaks up from fluid absorption.

In wounds that show signs of clinical infection, a more frequent wound inspection is advised. In these cases, an appropriate systemic treatment should be considered.

Storage

Cutimed® Sorbact® Hydroactive should be stored in dry conditions below 25 °C/77 °F. Store the dressing flat.

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