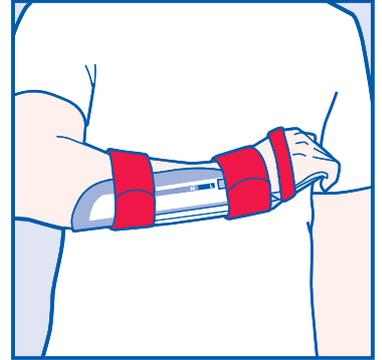
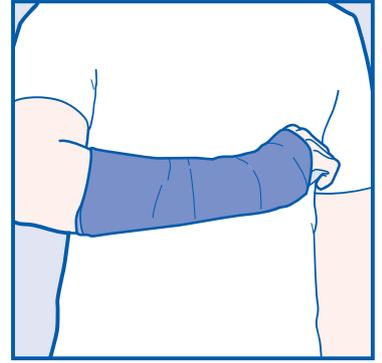
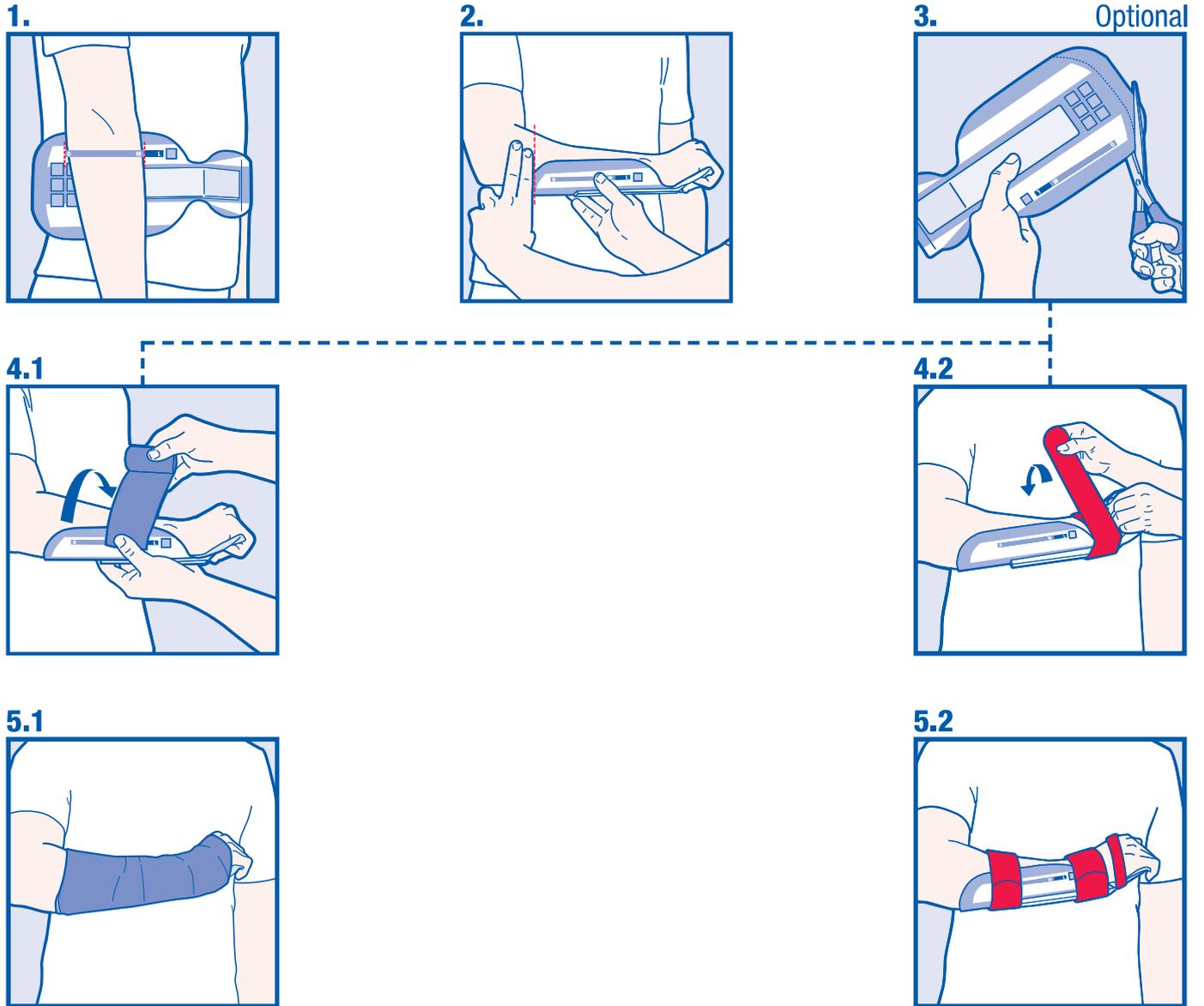
Delta-Xpress™

- (DE) **Gebrauchsfertige Handgelenkschiene**
(EN) **Ready to use wrist splint system**
(FR) **Système d'attelle de poignet prêt à l'emploi**
(ES) **Férula para muñera lista para usar**
(PT) **Ortótese para pulso pronta a utilizar**
(IT) **Ortesi per polso pronta per l'uso**
(NL) **Gebruiksklare polsspalk**
(SV) **Bruksfärdig handledsskena**
(NO) **Bruksklar håndleddskinne**
(DA) **Håndleddsskinne, "Klar-til-brug"**
(FI) **Käyttövalmis rannelasta**
(ZH) **即用型手腕夹板系统**
(JA) **すぐに使える手首用副木**
(AR) **جهاز جبيرة الرسغ الجاهز للاستخدام**

Delta-Xpress™

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Ready to use wrist splint system

Description

Delta-Xpress™ is a ready to use splint system for the immobilization/stabilization of the wrist joint. The pre-shaped splint system can easily be adjusted to the patient's body size and shape. The splint is secured to the limb with a bandage or with hook and loop fasteners. If fixed with a bandage, it is recommended to use a cohesive bandage. Because of its pre-shaped, anatomical form, the splint is fast & easy to apply. It is made of lightweight material that offers required immobilization/stabilization.

With its 100% x-ray translucency, the splint can be kept on the limb during x-ray examination.

Indications

Delta-Xpress™ is used for the short-term immobilization/stabilization of the wrist joint after surgery or after injuries of the lower arm and wrist joint such as fractures, wounds, sprains, strains, dislocations and burn injuries.

It is particularly recommended for

- Post-traumatic emergency immobilization/stabilization
- Primary care of soft tissue injuries
- Post-operative immobilization/stabilization
- Temporary immobilization/stabilization of the wrist before synthetic cast, synthetic splint or Plaster of Paris is applied
- Temporary immobilization/stabilization during X-ray examination.

Precautions

Any wound in the splinted area should be dressed prior to application of the splint. If necessary, use additional padding to cover bony prominences and prevent pressure points.

It is recommended not to use the product at temperatures higher than 40° C.

It is recommended not to use the product at a relative humidity higher than 75%.

Application Instructions

The term and use of the product is the responsibility of a qualified medical professional.

1. The width of the forearm indicates which size of splint to use. Measure length of the splint by holding the size scale of the wrist splint at the widest part of the forearm (fig. 1)
2. Hold the splint in the intended final position. In the correct final position the forearm as well as the carpals have full contact with the splint. With a 90° flex in the elbow, the space between the proximal end of the splint and the elbow crease should be 2 fingers wide at minimum (fig. 2).
3. If the splint is too long, adjust the length by cutting the proximal side of the splint with standard scissors (fig 3).
4. If desired, apply a stockinette on the limb and/or padding on areas of bony prominences to avoid pressure points.
5. Position splint on the limb as described in application step 2 (fig. 2) and secure with bandage or hook and loop fasteners.