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Cutimed® HydroControl

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Moisture-balancing, absorbent hydropolymer dressing

Description

Cutimed® HydroControl is a sterile, absorbent hydropolymer dressing that controls wound moisture conditions and supports optimal wound healing of dry and low to moderately exuding chronic wounds. It consists of a semi-permeable polyurethane film and a skin-friendly, absorbent hydropolymer adhesive layer with pores. The hydropolymer layer of the dressing protects the wound from drying out and creates a suitable environment for moist wound healing by absorbing exudate. The partial hydration of the dressing aids the process of autolytic debridement. The polyurethane film provides a low-friction, highly breathable, liquid-impermeable barrier to bacteria and viruses. The pore structure makes the dressing particularly flexible. The high moisture vapour permeability (MVTR) of the pores reduces the risk of build-up of fluid and maceration.

Indications

Cutimed® HydroControl is indicated for dry to moderately exuding chronic and secondary healing wounds, such as leg ulcers (venous, arterial and mixed aetiology), pressure ulcers and diabetic foot ulcers. Cutimed® HydroControl is suitable for use under compression therapy.

Contraindications

- Cutimed® HydroControl is not indicated for surgical implantation, for treatment of heavily exuding wounds, to control heavy bleeding, for third-degree burns or for the placement inside deep cavities or sinuses.

Precautions and safety information

- Cutimed® HydroControl is for single use only. Do not use if the package is already open or damaged as the sterility of the dressing is guaranteed only when the package is unopened and undamaged prior to use.
- Cutimed® HydroControl may be used on an infected wound in addition to antimicrobial therapy under medical supervision.
- Do not use on patients with a known sensitivity to glycerine or hydrogels. In case of intolerance, remove the dressing and gently cleanse the wound with water or sterile saline solution.
- The dressing may be moistened with sterile saline solution for easier removal.
- Should gel fragments remain in the wound bed after removing the dressing, these can be removed by irrigation with sterile saline solution or water.
- During the body's normal healing process, autolytic debridement (i.e. removal of non-viable tissue from the wound) could initially make the wound appear larger. If the wound continues to grow larger after the first few dressing changes, consult a healthcare professional.
- Because Cutimed® HydroControl creates an environment that favours the growth of new blood vessels, the delicate newly formed blood vessels may occasionally produce blood stained wound fluid. If excessive bleeding occurs, the dressing may be used together with an alginate wound dressing.
- Monitor the wound carefully. The dressing may remain in situ for up to seven days, depending on the wound condition. The total duration of contact (i.e. where the treatment involves consecutive application of individual dressings) is 30 days.
- Appropriate supportive measures such as compression in the case of venous leg ulcers or pressure-reducing measures in the case of diabetic foot syndrome and pressure ulcers, systemic antibiotics etc. should be used where necessary.
- Do not use the product after the expiry date.
- The dressing tends to swell upon contact with water, e.g. when bathing or showering. Therefore, it is recommended to reduce showering times to a minimum when being worn for longer periods.
- Data is not available that supports the use of topical medicinal preparations in conjunction with Cutimed® HydroControl.
- Do not use when receiving radiation therapy.
- For external use only.

Instructions for use

Preparation

- Cleanse the wound according to wound management protocol.
- The wound can be prepared by irrigation and drying the surrounding skin, as needed.
- Select one or more dressings so that the dressing overlaps the wound site by at least 2 cm. The dressing can be cut to size, if necessary.

Application

- Remove the dressing from the package.
- Peel away the lower, transparent carrier to expose the adhesive surface.
- Hold the dressing by the white applicator tab and apply with the adhesive surface towards the wound.
- Peel away the white tab and gently smooth the dressing.
- Carefully peel away the upper carrier at the red stripes.
- A secondary dressing can be used for fixation, if necessary. Cutimed® HydroControl is compatible with gauze bandages, elastic bandages, transparent film dressings and fixation strips.
- Inspect the wound frequently. Change the dressing as frequently as required by the wound condition.
- Cutimed® HydroControl may remain in situ for up to 7 days, depending on the patient condition and level of exudate.

Removal

- Carefully remove any fixation dressings according to wound management protocol.
- Gently remove Cutimed® HydroControl from the wound site and discard.
- Irrigate the wound with a suitable wound cleanser before inspecting the wound and applying a new dressing.
- Inspect the wound thoroughly when changing the dressing. Consult a healthcare professional if any of the following occur: signs of infection (increased pain, increased redness, wound drainage); excessive bleeding; a change in wound colour and/or odor; irritation (increased redness and/or inflammation); sensitivity (allergic reaction); no signs of healing.
- Make sure to remove Cutimed® HydroControl prior to radiation therapy.
- Do not reuse the dressing or re-sterilise – there is the risk of cross-contamination.

Do not use if the package is damaged. Protect from moisture and store dry at temperatures below 25°C/77°F.

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