# Therapietische Therapy Tables Therapietafels Terapiborde Tableaux Thérapeutique Tavoli terapeutici

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CH-REP. CH-Importeur: INNOVAhelp GmbH, Thunstr. 5a, CH-3110 Münsingen

## 01. Purpose and Indication

These instructions for use are aimed at employees, carers and therapists in the elderly care, hospital and rehabilitation sectors. It exclusively serves as a storage surface, which is fixed on the arm rests of a wheelchair. The therapy table is pushed over the height and width adjustable mounting brackets as well as the on the wheelchair arm rests, which need to be permanently mounted on the wheelchair. With the help of knurled nuts the therapy table is firmly clamped on the wheelchair. The ideal size of the therapy table is selected according to the seating width of the wheelchair.

## 02. Contraindication

The therapy table is unsuitable for all applications that deviate from the intended use and can lead to dangers. Please note that assembly is only possible on wheelchairs with armrests.

## 03. Safety Instructions

- Instruction manual: this operating instructions must be read before using the therapy table.
- Use and assembly: the therapy table is only intended for the use on wheelchairs with arm rests. It may only be mounted by qualified personnel. before each use the check the tightness of the table.
- Adjustment: there is a risk of jamming and crushing due to too tight or improperly adjusted therapy tables.
- Load capacity: observe the maximum load-bearing capacity! There is a risk of breakage! See section: „Technical Data“.
- Driving with mounted table: while driving remove all items from the therapy table! There is a risk of injury and burning caused by falling objects, for example: hot food or beverages.
- Collision when driving: there is a risk of injury through possible collisions while driving with a mounted table (pinching, squeezing).
- Usage in vehicles for handicapped: the therapy tables are not approved for the use in vehicles for handicapped. It may result in serious injury. Remove the therapy table and store it securely before using the vehicle.
- Fixation: the fixation of a disabled person through the therapy table may be an illegal measure that restricts liberty. Pay attention to the legal situation in your country/city!
- Breathlessness: a breathing limitation of the wheelchair user may occur when the therapy table is too tight adjusted or if the user gets tipped upside down forward. Do not leave the wheelchair user unattended!
- These instructions for use are part of the therapy table and must be handed over when the therapy table is passed on.
- If serious incidents occur in connection with this product, the incidents must be reported to both Rolko Kohlgrüber GmbH and the competent authority of the EU member state in which the user or patient is established.

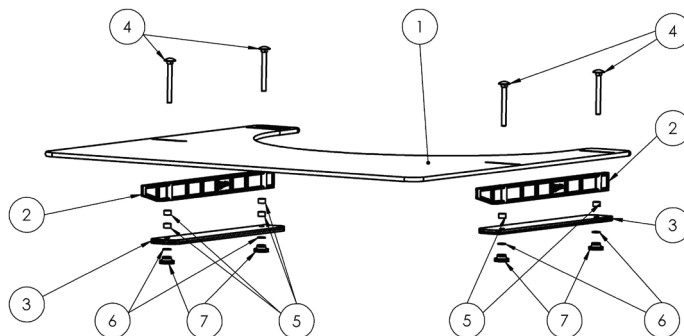
## 04. Assembly

Please, read this operating instructions carefully and check the product for possible transport damages. The therapy tables must only be assembled on wheelchairs with arm rests. The correct tightness needs to be checked after the assembly.

1. Remove the protective foil from the therapy table. Connect the therapy table (1), spacing strips (2) and ledges (3) with the screws (4). The tapered support side of spacer has to point to the front (shorter side points to the back). The strip must not overlap.
2. In order to acquire an optimal height adjustment, according to the arm rest height, please put the distance sleeves (5) on the screws.
3. Put the retaining washers (6) and pull the knurled nuts (7) for now lightly loose. Put the therapy table on the arm rests. Push the both mounted angle supports together so that the therapy table fits securely on the arm rests of the wheelchair. Please make sure that the spacing strips (2) are mounted parallel to the outer edge to avoid the risk of breakage. Always use all distance sleeves as well as spacing strips in order to avoid protruding screw threads.
4. Now tight the knurled nuts (7) and attach the therapy table to the arm rests. Please note that - if possible - the screw thread needs to be completely covered in order to prevent injuries, that may be caused by protruding screws. If necessary please use additional flat washers.
5. After the assembly please check the tightness of the table before using it.

Retainer assembly for the models:  
TP-L1, TP-L2 und TP-L3

Retainer assembly for the models:  
TP580-31, TP630-31, TP700-31 und TP650-31MK

## 05. Cleaning and Disinfecting

Clean the therapy table with lukewarm water with the addition of a neutral detergent. Use a soft, lint-free cloth for this. Do not use aggressive cleaning agents, solvents or hard brushes etc. Not resistant to: hot water and hot steam, acetone, halogenated hydrocarbons such as chloroform, dichloromethane, concentrated acids and alkalis.

## 06. Maintenance

Worn out or damaged parts need to be replaced immediately. Pay attention specially regarding the crack formation in the tabletop!

## 07. Re-use

The product is suitable for reuse. Due to environmental influences, UV radiation, mechanical stress as well as natural aging or careless handling, premature wear can be caused, over which the manufacturer has no direct influence. Basic cleaning is required for re-use, see section 5: „Cleaning and disinfection“

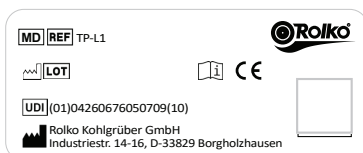
## 08. Disposal

Disassemble the therapy table as much as possible, the individual parts can then be disposed of with household waste.

## 09. Technical Data

Item no.: TP-L1, TP-L2, TP-L3, TP580-31, TP630-31, TP700-31, TP650-31-MK  
Suitable for arm rest height: 25 - 31 mm  
Max. load capacity: 10 kg

## Label

Manufacturer

Follow the instructions

Production date

Item no.

Batch

Medical device

Unique device identification

As an accessory for medical products of class 1, this product has a CE mark in accordance with regulation (EU) 2017/745 (MDR) and thus complies with all German and European safety requirements, that apply to the product. The product fulfils the corresponding sections of standard specification DIN EN ISO 21856. The EU declaration of conformity is available for download at [instructions.rolko.com](https://instructions.rolko.com).

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Reprint, also in extracts, only with prior permission by the editor. All rights reserved. The delivered product may differ from the illustrations in this manual. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen