Klappbare Therapietische Folding Therapy Tables Opklapbare therapietafels Sammenklappelige terapiborde Tablettes thérapeutiques escamotables Tavoli terapeutici pieghevoli

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01. Purpose and Indication

These instructions are aimed at employees, carers and therapists in the elderly care, hospital and rehabilitation sectors. The foldable therapy table is only suitable for use on wheelchairs and seat shells or seat shell bases. It hereby serves for resting the arms and as storage tray during meals, work, therapy or game play. The therapy table is fitted to the wheel chair's frame under the arm rest by using a special fixing system and can be folded to the side. It is available in three different versions, with a half table top (TP300-KLP-3) and with a full table top (TP600-KLP-3, TP620-POL-KLP-3). All models have a folding holder and are therefore usable with various seat widths. The folding therapy tables lie on the arm rests (for TP300-KLP-3 on one side) and can be fitted on the left or on the right.

02. Contraindication

The folding therapy table is unsuitable for all applications that deviate from the intended use and can lead to dangers. Please note that assembly is only possible on wheelchairs and seat shells or seat shell bases with armrests.

03. Safety Instructions


- Please read and mind the operating instructions.
- Application and assembly: The therapy table is only intended for application with arm rests. It must be fitted by qualified personnel only. The fixed position of the table must be checked before use.
- While folding or unfolding the therapy table make sure that it is completely folded, until the final position is reached. It is not allowed to lay down the table top on the push rim. This applies specially for wheelchairs with an additional electric motor, which may cause body injuries, damages to the wheelchair and the therapy table when the actuator is operated.
- Setting: There is danger of injury by movable parts, by wrongly-adjusted tables and by loose locking screws.
- Load capacity: Please mind the maximum load capacity. There is danger of breaking. See paragraph 'Technical Data'.
- Driving with table: Remove all objects from the therapy table! There is danger of injury through falling objects and danger of burns through hot food and beverages for example.
- Collisions while driving: There is danger of injury through possible collisions while driving with the table. Moreover, the overall width of the wheel chair changes through folding the table to the side. There is danger of collisions with furnishings etc.
- Usage in disabled transport vehicles: The therapy tables are not authorised for the use in disabled transport vehicles. There is danger of severe injuries. Remove the therapy table and pack it safely in the vehicle.
- Difficulty in breathing: There might be difficulties in breathing if the table is set too tight or if the disabled person tips to the front.
- Do not leave the user unattended and explain the user about the risks!
- Regarding the version TP620-POL-KLP-3: Avoid prolonged exposure to sunlight and follow the cleaning and disinfection instructions in section 05.
- The operating instructions are an integral part of the therapy table and must be handed over when passed on.
- In the event of serious incidents in connection with this product, these are the Rolko Kohlgrüber GmbH and the person responsible
- To report the authority of the EU Member State in which the user is established.

04. Assembly

Before assembling the product, the operating instructions is to be read carefully. Check the article for possible transportation damages and for completeness. Assembly is only allowed for wheel chairs with arm rests. The proper setting has to be ensured after assembling the product. When delivered, both articles are prepared for the left arm rest.

Assembly for the left side:



1. The holder (1) is removed from the rods (2) and fixed to the wheel chair under the arm rest on the left by using the rods' screw connections. Only for the model TP600-KLP-3: remove the protective film from the table top (3).
2. The table top (3) is fixed on the linkage adapter (2) by using the M6 screws (6) and nuts (7).
3. The rods (2) are pushed into the holder (1).
4. Loosen the swivel arm's locking screw (5) and adjust the length of the arm: TP300-KLP3: the tabletop should rest securely on the armrest of the wheelchair. TP600-KLP3, TP620-POL-KLP-3: the table top should be placed in the middle of both armrests of the wheelchair. Fasten the locking screws (5).
5. Set the table to its right position by shifting the rods (2) in the holder (1) and secure it by fastening the star knobs (4).
6. For therapy table TP300-KLP-3 (image 1): Set the stop screw (8) so that the table top is horizontal.
7. Check the safe position before use.
8. If necessary, use screw #11 to adjust how easy or difficult it is to turn/fold the table top.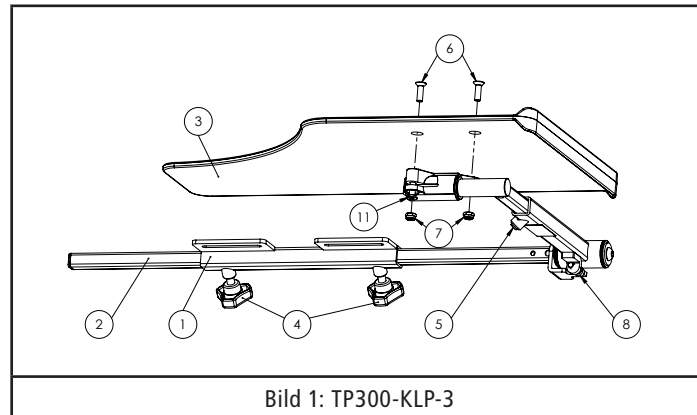

Conversion for assembly on the right side:

The TP300-KLP-3 is shown in the pictures, the modification of the TP600-KLP-3 and the TP620-POL-KLP-3 is identical.

1. For disassembly, mind image 2. For assembly, mind image 3.
2. The table's rods (2) are removed from the holder (1).
3. The holder is fixed under the right arm rest of the wheel chair.
4. The table top (3) is removed from the rods.
5. The screws (9) and (10) are removed completely and the locking screw (5) is loosened. Afterwards, the rods can be disassembled into the parts 2.1, 2.2, 2.3 and 2.4.
6. The parts 2.1, 2.2, 2.3 and 2.4 of the rods are then put together minding image 3 and the screws (9) and (10) including washers are tightly fastened. The swiveling arm's telescope has to be lightly fixed by using the locking screw (5).
7. Connect the table top with the rods by using screws (6) and nuts (7) .
8. Then, follow the assembly instructions of „assembly for the left side“ starting at step 3.
9. Check the safe position before use.

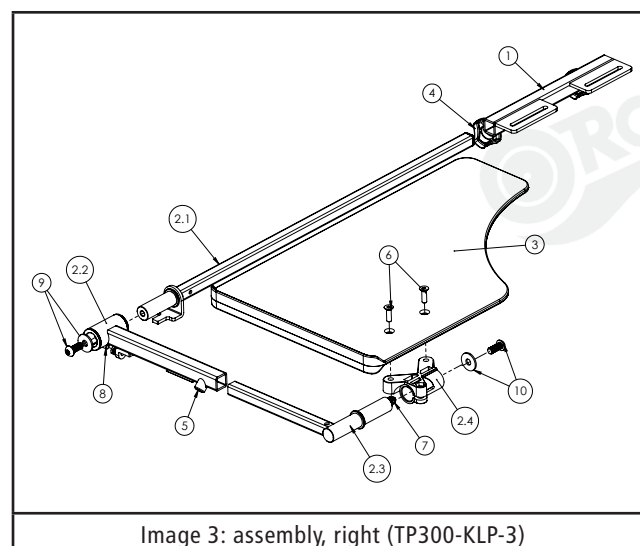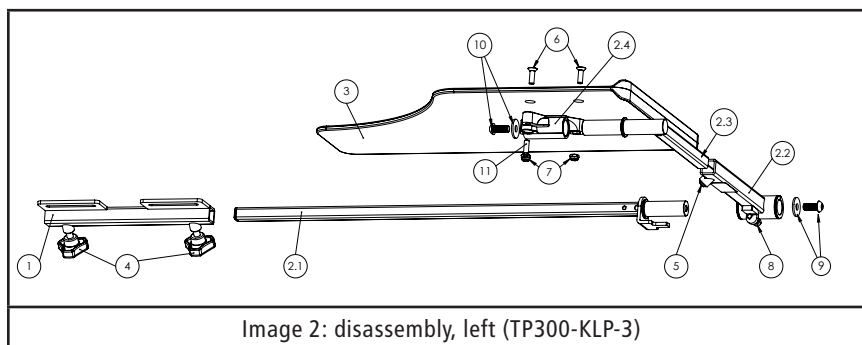Instruction Manual

05. Cleaning and Disinfecting

Clean and disinfect the therapy table before using it for the first time and before each subsequent use. Disinfect the table top regularly, but at least before each change of patient.

• Care and cleaning

For easy cleaning, the therapy table can be wiped with a damp cloth, if necessary with the addition of a mild cleaning agent. Use lukewarm water and, if necessary, non-abrasive, pH-neutral commercial household cleaners. Please also note the following point Disinfection / Sterilization. The application should only be done with a towel.

• Disinfection / sterilization

We recommend the cleaning agents and disinfectants listed below for cleaning the therapy table after consultation with their manufacturers. It may only be used in accordance with the manufacturer's specifications! Observe the relevant product information, particularly regarding the dosage and possible hazards when combining with other agents. Before using other cleaning agents or disinfectants, please always consult the manufacturer of the agent. Due to the large number of cleaning agents and disinfectants available on the market, as well as constant innovations, we cannot provide a comprehensive guarantee for compatibility. If in doubt, contact the sales partner or manufacturer, since the use of harmful substances can lead to premature wear or material failure and thus cause hazards.

TP300-KLP-3 and TP600-KLP-3: From the area of formaldehyde and/or other aldehydes or derivatives: Aldasan 2000, B5, Budenat Acute D 441, Desifor Protect, Desomed Perfect, hygienicVIRUZID, Kohrsolin extra, Lysoform, Lysoformin, Melsitt, Minutil, Nüscosept. Per compounds: Apesin AP100, Dismozon plus, Dismozon pur, Incidin active, Perform, Terralin paa, Wofasteril. From the field of phenol or phenol derivatives: AMOCID.

TP620-POL-KLP-3: From the field of biguanides: Incidin Plus. From the area of per compounds: Apesin AP100, Dismozon plus, Dismozon pur, Incidin active, Perform, Terralin paa, Wofasteril. From the ammonium chloride sector: Mikrozyd sensitive liquid, cosiMed disinfectant.

The following ingredients must not be used, as they change the material properties and lead to premature wear, which can result in material failure and thus hazards: All phenols or phenol derivatives not mentioned in these instructions for use, as well as other aromatic ones hydrocarbons. Chlorine, organic or inorganic substances with active chlorine. Lyes. All formaldehyde and/or other aldehydes or derivatives not mentioned in these instructions for use. Additionally for TP620-POL-KLP-3: alcohols, benzene, all formaldehydes and other aldehydes.

06. Maintenance

Worn-out or damaged parts are to be changed immediately. Especially a possible crack formation in the tabletop is to be monitored.

07. Re-use

The product is suitable for reuse. Due to environmental influences, UV radiation, mechanical stress as well as natural aging or careless handling, premature wear can be caused, over which the manufacturer has no direct influence. Basic cleaning is required for re-use, see section 5: „Cleaning and disinfection“


08. Disposal

Disassemble the therapy table as much as possible. Small and plastic parts can be disposed of with household waste. Larger metal parts are accepted by recycling.

Technical Data

Item No.:	TP300-KLP-3, TP600-KLP-3, TP620-POL-KLP-3
Measures Table Top:	TP300-KLP-3: 295x450x8 mm, TP600-KLP-3: 600x450x8 mm, TP620-POL-KLP-3: 620x450x37 mm
Own weight:	TP300-KLP-3 2,6 kg, TP600-KLP-3 4.6 kg, TP620-POL-KLP-3 3,6 kg
Material:	PC, PA6, PVC, PU, POM, PE, galvanized steel / painted black
Load Capacity:	max. 10 kg

Label

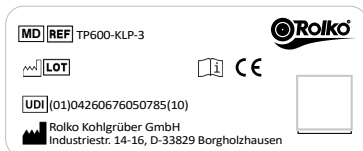Manufacturer

Follow the instructions

Production date

Item no.

Batch

Medical device

Unique device identification

As an accessory for medical products of class 1, this product has a CE mark in accordance with regulation (EU) 2017/745 (MDR) and thus complies with all German and European safety requirements, that apply to the product. The product fulfils the corresponding sections of standard specification DIN EN ISO 20417. The EU declaration of conformity is available for download at instructions.rolko.com.

Operating Instructions: Folding Therapy Tables / 2024-08-06 / Rev. 16

Basis-UDI-DI 426067605TPKL000000000WVY

© Rolko Kohlrüder GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen, Germany

Reprint, also in extracts, only with prior permission by the editor. All rights reserved. The delivered product may differ from the illustrations in this manual. All technical data are characteristic and subject to construction and manufacturing conditioned tolerances.

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© Rolko Kohlgrüber GmbH, Industriestr. 14 - 16, 33829 Borgholzhausen
CH-REP. CH-Importeur: INNOVAhelp GmbH, Thunstr. 5a, CH-3110 Münsingen