The natural way

SISSEL® SIT RING

- | Gebrauchsanweisung
- | Instructions for use
- | Mode d'emploi
- | Gebruiksaanwijzing
- | Instrucciones de uso
- | Istruzioni per l'uso

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General information

Read the following instructions for use carefully and follow the safety instructions and warnings. The SISSEL® Sit Ring is a ring-shaped cushion that enables comfortable and pressure-relieving sitting on all seating. The product may only be used in accordance with these instructions for use. SISSEL® GmbH accepts no liability for the use of the product outside the applications described in these instructions of use. The user manual must be kept carefully and within easy reach, and must be enclosed if the product is passed on. Finally, the regulatory requirements applicable to this product in the respective country of use must be taken into account.

Safety instructions

- ! Check the condition of the product before each use. In case of damage, the function of the product and safety is no longer guaranteed.
- ! Do not use the product in the immediate proximity of heat sources (e.g. radiators, hotplates, open flames, flying sparks) and avoid direct sunlight.
- ! Keep the area of use free of sharp or pointy objects.
- ! Use the product only on dry, non-slip, straight surfaces, e.g. on seats of chairs, stools or benches.
- ! Use only accessories recommended by the manufacturer.

The product and its accessories are no toys and should be kept out of the reach of children.
Do not allow children to play with it.

Product information

Scope of delivery

- 1 x Sit Ring round or oval with skin-friendly terry cloth cover, 1 x instruction for use

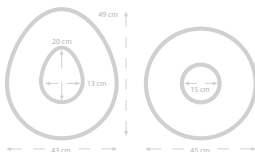
Technical data

Dimensions (L x W x H): oval: approx. 49 x 43 x 8 cm
round: Ø approx. 45 cm x 8 cm

Material: Polyurethane foam

Material Cover: Terry cloth (80 % cotton, 20 % polyester)

Max. User weight: max. 125 kg

Due to the production process, a slight yellowing of the foam and small air pockets may occur. This does not affect the function and stability of the product.

Note: Remove protective film from foam before first use.

Intended use

The SISSEL® Sit Ring provides pressure-relieving support in everyday sitting at home or at work. On all seats, the SISSEL® Sit Ring provides local pressure relief in the buttocks, rump, perineal and anal areas.

Indication

Local pressure relief for painful conditions in the buttock region, e.g. after operations in the perineal area, anal lesions, coccyx pain, prostatitis, cysts, haemorrhoids or during and after pregnancy.

Contra indication

None known.

Regulatory requirements

The current product of SISSEL® GmbH complies with the following regulatory requirements:

- Medical Devices Regulation (EU) 2017/745

All serious adverse events that occur in relation to the product must be reported immediately to the manufacturer and the competent authority.

Accessory

The product may only be used with the accessories intended for this purpose.

Optional accessories: Spare cover made of different materials, such as PU hygiene cover (liquid-tight) can be purchased additionally.

Usage

1. Check the packaging and the product for any damage upon delivery.

Never use a damaged product.

2. Check the product for integrity and contamination before each use.
3. Use the product as intended.

Note: Cleaning and, if necessary, disinfection measures may be required after use (see Care & hygiene).

Care & hygiene

The present product is suitable for reuse on additional persons. In general, the applicable national regulatory requirements for the hygienic reprocessing of medical devices must be observed and fault-free functioning must be ensured. SISSEL® Sit Ring cannot be disinfected. If used by more than one person, additional hygiene measures may be necessary. The Polyurethane foam core can be hand-washed with a mild detergent at lukewarm temperature. Then squeeze out carefully, do not wring out! Then leave to air dry on a grid

(place a towel underneath) for a sufficiently long time (approx. 1 week). Do not dry in the sun or on the heater. The terry cloth cover is machine-washable at 40 °C. Then air dry. Pay attention to the washing label. Store the SISSEL® Sit Ring in a clean, dry place out of direct sunlight. Sunlight can cause discolouration of the surface material. Discolouration does not affect the function.

Disposal & environment

The product and the accessories supplied with it can be disposed of with household waste. No special types of packaging are used. The respective valid disposal guidelines must be obeyed.

Warranty

The warranty is governed by the legal regulations. Any warranty is excluded if goods delivered by SISSEL® GmbH have been processed, treated or modified by others without consent or if the instructions for use have not been followed. The user of this product must comply with the regulatory requirements for medical devices applicable at the place of use.

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SISSEL GmbH