NOBASPLINT®

- D

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Ausgabestand: 17.03.2021 [Rev3]

GB Information for Use NOBASPLINT® synthetic fibreglass support bandage

Product Description:

NOBASPLINT® is a ready for use splint roll. It consists of 7 layers of fibreglass soaked with polyurethane resin. After its reaction with water, the resin hardens quickly. NOBASPLINT® excels with its adaptable plasticity at the extremities. The bandage has a low weight in spite of its high stability. It is air and water vapour permeable.

The cast is covered with a polypropylene nonwoven. This provides patients with an excellent wearing comfort.

Intended use and area of application:

NOBASPLINT® is a splint roll for immobilisation in cases of fractures, fissural fractures, contusions and distorsions. The product is applied by medical professionals.

Application instructions:

Wear nitrile examinations gloves during the application, for example NOBAGLOVE®NITRIL.

- Choose the desired size of NOBASPLINT® and remove the plastic bag.
- If NOBASPLINT® by the meter is used, cut off the desired length. Immediately reseal the plastic bag, or else the product will harden in the packaging.
- Submerge NOBASPLINT® for 5-10 seconds into water (18-25°C) and strongly squeeze it 2-3 times under water.
- Wrap NOBASPLINT® into a towel and squeeze out excess water.
- Apply NOBASPLINT® at the desired place and wrap it with an elastic bandage or possibly a cohesive bandage.

At the same time the splint should be moulded into a well-fitting shape. The setting process takes about 3-4 minutes, depending on the depth of submersion and the water temperature. Final stability is reached after about 20 minutes.

Contra-indications:

Do not apply the product in the case of a known allergy to the material.

Warnings and precautions:

- The reuse of single use devices may lead to a microbiological hazard. Treatment for reuse may adversely affect the performance of the product.
- Should serious incidents occur in relation to the device, they are, according to MDR (EU) 2017/745, to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Storage and transport:

At room temperature

Waste disposal:

In accordance with the requirements of the applicable local legislation and with the standards for the prevention of infections

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