NOBACAST®

- D** **Gebrauchsinformation**
NOBACAST® synthetischer Polyesterstützverband
- GB** **Information for Use**
NOBACAST® Polyester Synthetic Cast
- NL** **Gebruiksaanwijzing**
NOBACAST® synthetisch polyestersteunverband
- DK** **Brugsinformation**
NOBACAST® syntetisk polyester støttebandage
- F** **Notice d'utilisation**
Bandage synthétique de support NOBACAST®
- I** **Informazioni per l'uso**
Benda di supporto sintetica in poliestere NOBACAST®
- PL** **Informacje dotyczące użytkowania**
syntetyczny gips poliestrowy NOBACAST®
- ES** **Información de uso**
NOBACAST® vendaje de soporte sintético
- PT** **Informações de uso**
Ligadura de suporte sintética de poliéster NOBACAST®
- SV** **Bruksanvisning**
NOBACAST® syntetiskt polyesterstödförband
- CZ** **Informace k použití**
obvaz ze syntetického polyesteru NOBACAST®
- LT** **Naudojimo informacija**
NOBACAST® sintetinis poliesterinis tvarstis
- LV** **Lietošanas instrukcija**
NOBACAST® sintētiskā poliestera atbalsta pārsējs
- ET** **Kasutusalaane teave**
NOBACAST® sünteetilise polüestertugiside
- HU** **Használati útmutató**
NOBACAST® szintetikus poliészter sínkötés
- RO** **Informații de utilizare**
fașă de imobilizare din poliester sintetic NOBACAST®

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Ausgabestand: 16.03.2021 [Rev2]

Information for Use – NOBACAST® Polyester Synthetic Cast

Product description:

NOBACAST® is a synthetic support bandage made of woven polyester which is soaked with polyurethane resin. After its reaction with water, the resin hardens quickly.

Intended use and areas of application:

NOBACAST® provides rigid external immobilization for fractures, fissures, inflammations in the extremities, injuries and operations on the ligaments and further orthopaedic indications. NOBACAST® is suitable for use as a circular cast, splint, orthosis and also to provide patients with the braces technique. NOBACAST® is applied by medical professionals.

Application instructions:

1. Prepare the area to be immobilized with padding bandages or tubular paddings as required.
 2. Wear nitrile examination gloves during the application, for example NOBAGLOVE®Nitril. Open the packaging, remove the bandage and submerge it in water for 5-10 seconds (room temperature). Strongly squeeze the bandage 2-3 times under water as this leads to a quicker beginning of the chemical setting reaction. The more often you squeeze the bandage, the quicker it will harden. Please note that the colder the water is, the longer the bandage will take to harden.
 3. When applying the bandage, ensure that each wind overlaps the previous wind by one third to a half of the bandage width. Make sure the bandage is applied without wrinkles. NOBACAST® has good longitudinal elasticity and exceptionally good lateral elasticity to facilitate wrinkle-free application.
- **Cleaning NOBACAST®:**
NOBACAST® may be cleaned by wiping with a damp cloth.

- **Removing NOBACAST®:**
NOBACAST® may be removed using conventional cast cutters or cast scissors.

Contra-indications:

Where there is a danger of hematoma, edema or increased swelling (for example if used for the primary treatment of a fracture), NOBACAST® may not be used as a circular bandage. After the bandage has set, it must be split completely. Do not apply the product in the case of a known allergy to the material.

Warnings and precautions:

- If NOBACAST® is removed with a saw-type cutter, the formation of dust may occur. It is therefore recommended to use a filtering half mask as protection against particles (such as NOBAPROTECT®).
- Make sure the blade is regularly cleaned to prevent solidified resin, which might cause an increased heating effect when the cast is opened.
- The reuse of single use devices may lead to a microbiological hazard. Treatment for reuse may adversely affect the performance of the product.
- Should serious incidents occur in relation to the device, they are, according to MDR (EU) 2017/745, to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Storage and transport:

At room temperature

Waste disposal:

In accordance with the requirements of the applicable local legislation and with the standards for the prevention of infections

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