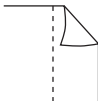
**PHYSIO  
CONTROL**

**MD** !USA Rx Only

**REF** 11996-000093

Important Information

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Važne informacije

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Oluline teave

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Informations importantes

Wichtige Informationen

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Viktig information

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**AUS** **Stryker Australia Pty Ltd**

8 Herbert Street, St Leonards  
NSW 2065 Australia

**EC** **REP**

**Stryker European Operations Limited**

Anngrove, IDA Business & Technology Park  
Carrigtwohill, Co. Cork, T45 HX08 Ireland

PT00098024

# ENGLISH

## WARNINGS

Possible skin burns, or ineffective therapy can occur due to air pockets, damaged, or dried out electrodes.

- Apply so entire electrode adheres to skin.
- Do not reuse electrodes.
- Do not use if damaged or dried out.
- Do not open package until immediately before use.
- Do not use beyond Use By date.
- Do not crush, fold, trim, or store under heavy objects.
- Replace electrodes after 25 shocks, 8 hours of continuous pacing or 24 hours on skin.

Possible fire, skin burns, or ineffective therapy

- Skin burns may appear with delivery of 100J shocks or after 30 minutes of pacing.
- Upon cessation of pacing, immediately remove or replace electrodes.
- Do not discharge hand-held paddles through electrodes.
- Do not allow electrodes to touch each other, ECG electrodes, lead wires, dressings, or other items on the patient's chest.
- Do not use near Magnetic Resonance Imaging (MRI).

Place electrodes away from implanted devices to prevent interference.

## IMPORTANT!

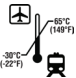
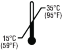
- Refer to your LIFEPAK device operating instructions for complete directions for use, indications, contraindications, warnings, precautions, potential adverse effects, and device compatibility.

- Use only with devices configured for QUIK-COMBO electrodes.
- Do not use with LIFEPAK 500, LIFEPAK 1000, LIFEPAK CR Plus, or LIFEPAK EXPRESS defibrillators.
- Before use, verify electrode pouch seal is not damaged.
- The user and/or the patient should report any serious product-related incident to both the manufacturer and the local regulatory authority, such as the competent authority of the European Member State, where the user and/or patient is established.
- Do not use alcohol or tincture of benzoin to prep the skin.
- Do not immerse in water or fluids, sterilize or clean.
- Stryker or its affiliated entities own, use, or have applied for the following trademarks or service marks: LIFEPAK, LIFEPAK CR Plus, LIFEPAK EXPRESS, QUIK-COMBO, and Stryker.

## Symbols

The symbols in the following table may be found on the electrode or its packaging.

Symbol	Description
	Consult Instructions for Use
	Dangerous voltage. Do not touch patient during defibrillation.
	Do not use this pediatric QUIK-COMBO electrode on LIFEPAK 500, LIFEPAK 1000, LIFEPAK CR Plus, or LIFEPAK EXPRESS defibrillators.
	Type BF applied part

Symbol	Description
	Do not reuse
	Use by date shown: YYYY-MM-DD
	Not made with natural rubber latex
	Manufacturer
	Date of Manufacture shown: YYYY-MM-DD
 	Do not dispose of this product in the unsorted municipal waste stream. Dispose of this product according to local regulations. Visit <a href="http://strykeremergencycare.com/recycling">strykeremergencycare.com/recycling</a> for instructions on disposing of this product.
	Recommended transport temperature range -30° to 65°C (-22° to 149°F). Storage at these temperatures is limited to seven days. If storage at these temperatures exceeds one week, the electrode shelf life is reduced.
	Recommended storage temperature range of 15° to 35°C (59° to 95°F).
	Keep away from sunlight
	Mark of conformity to applicable European regulations and/or directives

Symbol	Description
	Canadian Standards Association certification for Canada and the United States
<b>EC</b> <b>REP</b>	Authorized representative in the European Community
<b>AUS</b>	Australia sponsor address
<b>!USA</b>	For USA audiences only
<b>Rx Only</b>	By prescription only
<b>MD</b>	Medical device
<b>PN</b>	Part number
<b>QTY</b>	Quantity
<b>LOT</b>	Lot number
<b>UDI</b>	Unique device identifier
<b>REF</b>	Catalog number
<b>PAT</b>	Visit <a href="http://strykeremergencycare.com/patents">strykeremergencycare.com/patents</a> for patent information

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**Physio-Control, Inc.**