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User manual / Istruzioni per l'uso e il montaggio / Mode d'emploi
et de montage / Monterings- og betjeningsvejledning /
Montagehandleiding en gebruiksaanwijzing

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EN User manual

Intended Use:

The Rubber Crutch/Cane Tips with Spikes is a medical device and is mounted at the lower end of forearm crutches or metal sticks. It protects the walker from wear and tear and can make it safer to place the walking aid on icy, frozen or snow-covered ground by using the spiked cap.

Indication:

Use of walking aids with limited walking ability on weather-related smooth surfaces such as frost, ice, snow or similar.

Contraindication:

Insufficient physical or mental abilities for use, unsuitable undergrounds.

Description / Versions:

The Tip with Spikes consists of a crutch capsule and a cover cap with integrated spikes, which can make it safer to put the walking aids on the above-mentioned surfaces. The crutch capsule can be used without the cover cap like a conventional capsule, before entering weather-related smooth, frozen or similar surfaces, the cover cap is attached. There are different models or sizes, the selection of the appropriate (replacement) capsule for the existing walking aid must be done together with authorized, trained personnel. The testing, assembly and instruction in use may only be carried out by the specialist personnel. The model is designed for domestic use, must only come into contact with intact, uninjured skin and must not be used across patients. Any use other than that mentioned above is not intended. Sundo GmbH is not liable for any resulting damages, the risk is borne exclusively by the user. In the event of serious incidents related to this product, please inform the manufacturer or the competent authority.

Before first use / Maintenance:

First, read the instructions for use completely and keep them until the end of the product life. Before first use and each use, please check by visual inspection whether the product is free of defects (damage, wear and tear, etc.) and firmly mounted. If there are signs of damage or wear, the crutch capsule must not be used (any longer). Further maintenance is not required if the safety instructions are observed.

Assembly:

The existing crutch capsule is removed and replaced by the crutch capsule with spikes. Please do not use sliding or other fatty agents (risk of slipping). The walking aid can be used with the crutch capsule as usual, before entering icy, frozen or similar, the cover cap with the spikes is placed on the capsule to increase safety. Pay attention to the correct fit to ensure functionality. If the cap is no longer needed, it can be removed again.

Cleaning and disinfection:

Do not use sharp or abrasive objects or detergents for cleaning. Use only mild, commercial household cleaners and soft cloths or sponges. Do not use greasy or oily cleaning agents, risk of slipping and falling! Be careful while cleaning spikes, risk of injury! All parts can be treated with commercially available, chlorine-free disinfectants.

Important: please pay attention to the manufacturer's application instructions.

⚠ Safety:

- ▶ Caution, risk of injury! This product is not a toy; do not leave children unattended with it!
- ▶ To minimize the risk of slipping or falling, place the crutch or cane as vertically and completely as possible on the ground.
- ▶ Be careful on smooth, slippery or wet surfaces! Adjust your walking behavior accordingly, otherwise there is a risk of falling!
- ▶ Use the spikes only on icy, frozen or snow-covered surfaces! Be careful with hard surfaces (icy sidewalks, concrete, etc.), there is a risk of slipping, increased attention is required! Remove the cap as soon as you enter normal floor, interiors, etc. Only use the spikes outdoors!
- ▶ Because of the intended use the spikes are very pointed, be careful when (dis)assembling and cleaning, there is a risk of injury!
- ▶ Use the crutch capsules with spikes only in the winter months / in the mentioned weather situations and then exchange them for the usual capsules. Check the capsules regularly for visible wear or damage and replace them regularly. If there are cracks, (also one-sided) wear on the tread and/or an insufficient profile, the capsules must not be used any longer! In these cases, contact your specialist dealer.

Warranty:

According to the legal warranty claims. From the warranty excluded are damage by inappropriate use, normal wear or use as well as defects, that affect the value or usability of the device only insignificantly. Technical changes, alterations and extensions and improper maintenance. Work lead to expiring the warranty and the product liability.

We reserve ourselves changes, which serve the technical improvement.

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