ALLEVYN[®] COMPLETE CARE

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Para pensos multistrie

ALLEVYN[®] COMPLETE CARE

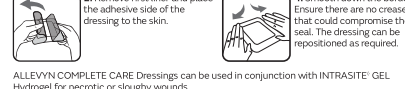
Description
 ALLEVYN COMPLETE CARE Dressings are silicone gel adhesive composite hydrocolloid foam dressings. The absorbent pads, a foam layer and lock away core made from super absorbent material. There is a discretion layer over the centre of the pad, the dressing is breathable and has a waterproof top film.

SmithNephew

Mode of Action
 ALLEVYN COMPLETE CARE Dressings have a perforated silicone adhesive wound contact layer which allows exudate to pass through to the foam layer. The silicone adhesive layer is gentle, suitable for use on fragile skin and makes it easy to apply, reposition and remove. The foam layer absorbs exudate away from the wound and peri-wound and the ExULOCK[®] hyperabsorbent layer absorbs exudate and helps it to spread laterally across the dressing gradually, utilising the entire dressing area to lock in exudate, helping to control odour. The breathable top film allows evaporation and helps prevent bacterial contamination of the wound. Together the absorbent and breathable layers promote a moist wound environment which is optimal for healing of low to highly exuding wounds. The ExUMASK[®] Masking Layer visually masks unsightly exudate and provides a visual indicator of when the dressing needs to be changed.

Indications
 1. Wound management on shallow, granulating wounds, chronic and acute exudative wounds, full and partial thickness wounds including :
 - Leg Ulcers
 - Diabetic Foot Ulcers
 - Surgical wounds
 - Superficial and partial thickness burns
 - Skin graft donor sites
 - Skin tears
 - Fungus Ulcers
 - Queimaduras superficiais e de espessura parcial
 - Locais doadores de enxertos de pele
 - Quebras cutâneas
 - Lesões fúngicas

Instructions for use
 Cleanse the application site. Ensure surrounding skin is clean, dry and free from excess hair.

ALLEVYN COMPLETE CARE Dressings can be used in conjunction with INTRASITE[®] GEL Hydrogel for necrotic or sloughy wounds.

Observation : When used for pressure injury prevention, frequently inspect the skin for pressure damage by lifting back one corner, inspecting and then repositioning the dressing.

Removal : Lift one corner and slowly peel back until completely removed to avoid skin injury. Improper disposal of dressing and packaging can cause slips, contamination, and infection, dispose of them per local clinical guidelines.

Frequency of change
 During the early stages of treatment inspect the dressing frequently. Dressings can be left in place for up to 7 days. The diagrams below are for guidance only and the decision of when to change should be dependent upon clinical assessment and local protocols under the guidance of a healthcare professional.

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*Trade Marks of Smith-Nephew. Date of issue 11/2025. Revision 02. This product may be covered by one or more US Patents www.smith-nephew.com/patents 18602917 48487

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