bella
my
bella

HOW TO SANITISE IN MICROWAVE

Insert the cup in the case.

1


Fill the case
with water to half.
Don't cover.

2

Heat the cup in the microwave
at 850 watts for 5 minutes.
Take off the case. Be careful
the case can be very hot.

3

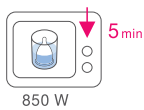HOW TO WASH

Insert the cup in the case.

1


Fill the case with hot or cold
water. Close the case's lid
and shake for a few seconds.

2

Open the case,
pour out the water
and remove the cup.

3

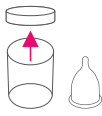menstrual set

DAY
& NIGHT


S

menstrual
cup
+
cleaner case

Choose the right size:

	S	M	L
Recommended for women	under 20 and first-time users	20-30	over 30
	Ø 42 mm H 58 mm 25 ml	Ø 42 mm H 64 mm 29 ml	Ø 46 mm H 68 mm 40 ml

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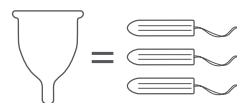Information for the Indian market: Bella Menstrual Cup, Size: S, Net qty: 1 Piece,
Store away from moisture, direct heat and sunlight. Date of Manufacture, Expiry
date, MRP Rs. (incl. of all taxes): see on the side. Country of origin: USA. Imported
and Marketed by: BellaPremier Happy HygieneCare Pvt. Ltd., No. 19/1 Madurai-
Dindigul Highway, Ambathurai, Dindigul District, Tamil Nadu - 624 302, India.
For feedback, contact our customer care executive at the importer's address.
Ph: 080 267 426 44, Email ID: customercare.in@tzm-global.com, Website:
www.tzm.in, WhatsApp: +91 728 900 13 21.

READ MORE

tzm-global.com
bella-global.com

PRODUCT AND PACKAGING
DESIGNED IN ACCORDANCE
WITH THE PRINCIPLES OF
ECO-DESIGN, LIMITING
THE NEGATIVE IMPACT
ON THE ENVIRONMENT.
MORE AT:
tzm-global.com
/sustainable-development

Read the leaflet
before use