



VAH

IHO

Disinfection wipes

# DESCOSEPT SENSITIVE WIPES

Alcohol-impregnated wipes for rapid disinfection

- Universal, comfortable and time-saving application
- Broad, rapid efficacy
- Especially suited for disinfection of medium-sized surfaces in near patient areas



Product description

DESCOSEPT SENSITIVE WIPES are alcoholic, ready to use wipes for disinfection and cleaning of medical devices surfaces, especially sensitive surfaces. DESCOSEPT SENSITIVE WIPES offer a universal and time saving application for the disinfection and cleaning of larger surfaces. The wipes can be extracted individually from the re-sealable and user-friendly flowpack.

DESCOSEPT SENSITIVE WIPES are based on an optimal combination of wipe and soaking solution. Therefore they offer effective, yet gentle and streak free disinfection within a short time.

Range of application

Acc. to Medical Device Directive (MDD): Rapid disinfection of non-invasive alcohol resistant medical devices. Especially suited for displays, keyboards and control surfaces of sensitive medical equipment.

Acc. to Biocidal Products Regulation (BPR): Rapid disinfection and cleaning of alcohol resistant medical inventory and all kinds of surfaces.

Spectrum of efficacy and contact time			30 s	1 min	2 min	3 min
<b>Bacteria and yeasts</b>						
application recommendation for surface disinfection: Bactericidal, yeasticidal*	VAH**	with mechanical action, dirty conditions		•		
application recommendation for surface disinfection: Bactericidal, yeasticidal*	EN 16615	with mechanical action, dirty conditions			•	
tuberculocidal (M. terrae)	EN 14348	clean conditions	•			
mycobactericidal (M. avium)	EN 14348	clean conditions		•		
<b>Enveloped viruses</b>						
active against enveloped viruses	RKI/DVV	incl HBV/HIV/HCV	•			
<b>Non-enveloped viruses</b>						
active against rotaviruses	EN 14476	clean conditions	•			
active against noroviruses (MNV)	EN 14476	clean conditions		•		
active against adenoviruses	EN 14476	clean conditions				•
<b>Additional test results</b>						
bactericidal	EN 13727	clean and dirty conditions		•		
yeasticidal (C. albicans)	EN 13624	clean and dirty conditions		•		

\*including phase 2 step 1 and phase 2 step 2 tests (quantitative suspension tests and quantitative carrier tests)  
 \*\*listed in the 5 min column of the VAH list

Application

Wipe surfaces with DESCOSEPT SENSITIVE WIPES until completely wet and allow to dry for the entire contact time.

Application notes

Shelf life after opening: 3 months.

Composition

100 g solution contain: 45 g ethanol



# DESCOCEPT SENSITIVE WIPES

Alcohol-impregnated wipes for rapid disinfection

## Expert opinions

We have extensive expert reports for all our products. Upon request we are glad to supply you with an expert report portfolio.


## Precautionary and hazard statements

Flammable liquid and vapour. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.

For professional use only.

Product	Package	Unit	Content	Size	REF
DESCOCEPT SENSITIVE WIPES	pack	54	10 wipes	20 x 22 cm	00-323DS-T010
	pack	12	80 wipes	20 x 22 cm	00-323DS-T080-01
Additional supplies	wall mount	-	1	-	00-902-FPH

CE 0482

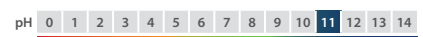

Use disinfectants safely. Always read the label and product information before use.

Dr. Schumacher is certified according to DIN EN 13485, DIN EN ISO9001, DIN EN ISO14001, BS OHSAS 18011 and has a validated environment management system according to EMAS and is a member of IHO, VCI, BAH, DGSV and of the DGKH. [www.schumacher-online.com](http://www.schumacher-online.com)

