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**DEUTSCH**

### HEINE® Modular+F.O. Laryngoskope

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**ENGLISH**

### HEINE® modular+ F.O. laryngoscope blades

	Please read these instructions carefully before using the HEINE® modular+ F.O.laryngoscope blades and keep them in a safe place for future reference.
---	---

#### Safety Information

**Range of applications:** HEINE laryngoscope blades are used only for endotracheal intubation. Only persons with training in endo-tracheal intubation are authorised to use them.

**Safety in use:** Only blades and handles with green identification marking should be used together. The laryngoscope blade should only be used with the appropriate light module (e.g. Mac 3, Mil 2). Bring the blade into the normal working position and check that the light switches on. Do not use the laryngoscope in the presence of a strong magnetic field. The bulb brightness can reduce during use if the rechargeable battery was not charged sufficiently before use. We recommend a minimum recharge time of 10 minutes for HEINE rechargeable batteries in HEINE chargers. Bring the blade into the normal working position and check that the light switches on. After some minutes' use the laryngoscope bulb will be hot. The laryngoscope may only be used by trained personnel.

**General safety:** To avoid cross-infection, the laryngoscope must be cleaned and disinfected or sterilized after each intubation, (see cleaning instructions). Conventional methods of cleaning, disinfection and/or sterilization are not known to eliminate Prions.

Where the instrument is subject to frequent use we recommend that a spare instrument and a small stock of replacement bulbs and batteries should be permanently available.

#### Instructions for use

##### Macintosh Blades

**Fitting the F.O. Light module:**

Push the module forwards into the blade module until it clicks into place. The light exit (1) must fit the window (2).

**Detaching the F.O. Light module:**

Hold the blade in one hand and push the module back and out of the blade module.

##### Miller Blades

**Fitting the F.O. Light module:** (only in blade modules the same size!)

Insert the short end of the light module into the opening in the heel of the blade module (3) and swivel towards and under the blade module upper part (4) **until it clicks into the working position (5).**

**Detaching the F.O. Light module:**

Hold the blade in one hand and push the end of the light module out of its working position with the thumb of the other hand and pull it out of the heel of the blade.

##### Functional test

**Before each intubation test the function of the laryngoscope.** Attach the blade to a handle and swivel the blade into its working position.

#### Care and maintenance

**Cleaning, disinfection and sterilisation**  
**Procedure after use**

Remove all traces of contamination immediately after use to avoid incrustation.

##### Manual cleaning / disinfection

Use only cleaning and disinfection products which are recommended by the manufacturer for use with fiber optics and plastics. Follow the manufacturers instructions regarding concentration and duration of use exactly. After this process, rinse the blade with de-mineralised water and ensure that no deposits remain on them.

**Warning:** Use of the wrong cleaning and disinfection products or methods can cause irreparable damage to the laryngoscope.

**Ultrasonic cleaning is not permitted!**

##### Automatic cleaning / disinfection

The laryngoscope can be cleaned or thermally disinfected at up to 93 °C in an automatic machine. Use only cleaning products, which are recommended for use with stainless steel and plastic. The rinsing procedure must ensure that no deposits are left on the product. Use only de-mineralised water for the final rinses.

##### Sterilization

Laryngoscopes have to be sterilized as required by national regulation and internal institutional standards. We recommend the sterilization with saturated steam at 134 °C for 4 minutes. Cycles of up to 18 minutes or up to 60 minutes in specified cases should not damage the metal part of the blade. There may be a reduction in light transmission with frequent / repeated long-term steam sterilization, particularly if the steam quality is not ideal.

It is the responsibility of the end user to ensure that the cleaning and sterilization is actually performed using appropriate equipment, materials, and personnel to achieve the desired result. This normally requires validation and routine monitoring of the process. Any deviation from these instructions should be evaluated for effectiveness and potential adverse consequences.

	<b>Observe Operating Instructions</b>
Duration of use:	continuous
Ambient conditions:	
In use:	10 – 40 <span> </span> °C
Storage:	5 – 45 <span> </span> °C
Relative humidity:	10% to 75%
Air pressure:	700hPa to 1060hPa
	Equipment is not suitable for use in the presence of flammable mixtures.
Nominal voltage:	see marking on lamp housing
Protection against electrical shock (BF type):	

**Applicable standards: DIN EN ISO 7376**  
**IEC60601-1: 1988 + A1: 1991 + A2: 1995**  
**(Medical Electrical Equipment Part1 – General Requirements for Safety)**

#### Warranty

We guarantee these laryngoscope blades for 5 years up to 4000 autoclave cycles of 134 °C / 5 min. from date of purchase. We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded. For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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