

## HEINE Anoscope / Proctoscope with proximal illumination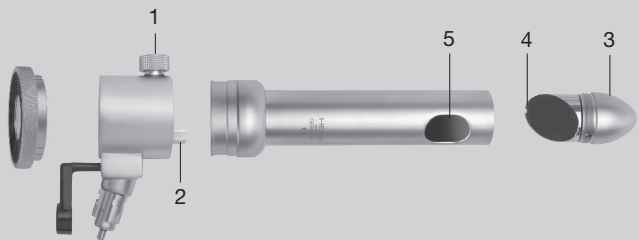

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49 (0) 81 52 / 38 - 0  
Fax +49 (0) 81 52 / 38 - 2 02  
E-Mail: info@heine.com · www.heine.com  
med 13596 1/2.12

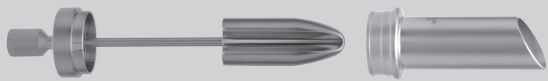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### Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries).

We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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## Instructions for use

### HEINE Anoscope / Proctoscope with proximal illumination

Read these instructions carefully before using the HEINE Anoscope/Proctoscope with proximal illumination and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

#### Safety information

**Range of applications:** HEINE Anoscopes and proctoscopes with proximal illumination are designed only for examination and treatment of the anal passage.

**Safety in use:** Use only light- or power- sources which are approved for use in a medical environment. Tubes with straight direction of view should only be inserted when an obturator is fitted.

#### Using the instrument

- Connect the instrument head to a handle.
- Insert the tube into the head as far as the stop.
- Rotate the tube until the viewing window is in the desired position. Secure the tube with the screw (1).
- Insert the obturator through the open instrument head.
- The cone of the obturator and the tube should be lubricated with a suitable lubricant.

The viewing window and swivel lens magnify by approx 2 x. With a short tube they can be used together to give ca. 3.5 x magnification.

#### Adjusting the mirror assembly

With tubes with a lateral viewing window, insert the mirror assembly (3) into the tube so that the groove (4) coincides with the pin (5). Screw in the mirror assembly as far as possible.

#### Changing the bulb

**Please note: The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.**

Unscrew the bulb from the head (2). Screw in a new bulb.

#### Use with a cold light source

The F.O. (fiber optics) head can be illuminated by any HEINE cold light source.

#### Cleaning / Disinfection and Sterilisation

##### Preparation at the point of use/ Initial Cleaning

Residues and other deposits must be removed immediately after use to avoid any residues drying on to the instrument.

##### Disassembly

The instrument must be removed from the light source for further preparation steps. The article must be disassembled to ensure a satisfactory cleaning result (see picture).

#### Tubes and Obturator

##### Manual disinfection and cleaning

For manual cleaning and disinfection, only those cleansing products which are approved for use on medical devices made of stainless steel, fiber optics and plastics may be used. The manufacturer's recommendations for use must be followed to avoid material damage / corrosion. After treatment, the instrument must be thoroughly rinsed with de-mineralised water to avoid surface residues.

##### Machine cleaning and disinfection / Thermal Disinfection

The article can be machine-cleaned and disinfected at up to  $\leq 93^{\circ}\text{C}$ . Only cleansing products which are approved for use on medical devices made of stainless steel, fiber optics and plastic may be used. The instructions of the manufacturer of both machines and detergents must be followed. The machine should be loaded correctly so that the cleaning procedure is successful. Rinsing procedures must ensure that no deposits are left. The final rinse should be carried out using de-mineralised water.

Ultrasonic Cleaning is not allowed for articles with fiber optics.

##### Sterilization

We recommend sterilization with saturated steam at  $134^{\circ}\text{C}$  for 5 minutes. Cycles of 18 minutes are allowed.

#### Instrument head

Instrument head can be treated by surface disinfection.

**Warning:** Non-suitable or incorrectly-used chemicals or preparation procedures can lead to irreparable damages to the article.

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